ESPAÑOL (AMERICA LATINA)

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ENGLISH (NORTH AMERICA)

ANDAFACIL 2 IN 1 ROLLATOR WITH SEAT

ANDAFACIL 2 IN 1 ROLLATOR WITH SEAT USER MANUAL

WARNINGS

Before making any use of the walker, make sure that the walker is in good condition. In case the product is damaged, do not use it. Users are responsible for the use of the upright walker.

- The brakes must be in locked position before sitting on the seat during rests
- Ensure that the Walker is fully opened
- Do NOT exceed the maximum weight capacity (300lb)
- Do NOT stand on the walker

DESCRIPTION OF THE ANDAFACIL WALKER

The Andafacil 2 in 1 Rollator with seat is made of resistant aluminum and it works both as a Walker and as a transpor chair

COMPONENT MATERIAL

Frame: Aluminum
Tires: Rubber
Handles: Polyurethane
Seat: Nylon and Polyester

PREPARATION FOR USE

Make sure to assemble the walker correctly and, if necessary, ask for the support of a doctor or nurse to adjust the appropriate height for the user.

SPECIFICATIONS

Model	ADCA2022
Width inside hand grips	198 lb / 50cm
Overall width	30.5 in /53.5cm
Overall length	77cm
Adjustable height	40.5-42.5 in / 103-108cm
Seat area	17.9 x 14.5 in / 45.5x37cm
Seat height	55cm
Front wheel	8 in PU) / 20.3cm
Rear wheel	12 in (PU) / 30.4cm
Net weight	22.7lb (10.3kg)
Weight capacity	300lb (136kg)

The Andafacil 2 in 1 rollator needs to be assembled before use. Please read the manual and familiarize yourself with the parts.

COMPONENTS

ANDAFACIL 2 IN 1 ROLLATOR WITH SEAT USER MANUAL

1. Install Handles

- Open and stretch the main frame.
- Insert the brake cable and handle into the frame tube.
- Press the orange button until you hear the click

2. Insert the front Wheel tube

- Insert the hexagonal screw in the hexagonal hole.
- Insert the washer five star know in the other side
- Tighten the knob.

3. Install rear Wheel sets

- Insert the tube of the rear Wheel and press the button to push in until you hear a click.
- Insert the brake cable into the frame tube.

4. Open the rollator

- Push on the seat from the edges until you hear a click.
- Doble check that the rollator is firmly secured and properly opened
- To fold the rollator, pull the plastic handle on the seat.

5. Install the stand-up handles

- Insert the handle tubes into the frame until you hear a click.

6. Attach the cane holder and shopping bag

- Push on the seat from the edges until you hear a click.
- Double check that the rollator is firmly secured and properly opened
- To fold the rollator, pull the plastic handle on the seat.

Foldable & Expandable Footrest

- Install the footrest to the main frame
- Rotate footrests depending on need

Adjustable Height Footrest

- Unscrew the knob
- Push the button to adjust height as needed
- Tighten back the knob after adjustments

Reversible and padded backrest

- Install backrest to the front or to the back to use as Walker or wheelchair as needed

INSTALL

RELEASE

CLEANING

The Andafacil walker should be cleaned with a damp cloth. Use a disinfectant cleaner if required. The walker must be completely dried before using it again to avoid operating risks.

MAINTENANCE

Over time the brakes may need adjustment. If the cables feel loose, adjust them the same way you adjust the brakes on a Bicycle. Make sure brakes and handgrips are properly always adjusted.

Maintenance must be carried out as follows:

WEEKLY

- Check the operation of the brakes
- Check that the tires are tight

PERIODICALLY

- Check the handles
- Make sure the screws are tight
- Make sure the tires are tight

Thanks for your purchase. We hope you enjoy your Andafacil walker.

Support us with a rating on the platforms and networks and you will receive a special gift from us

Visit the page or scan the QR code to get started
www.andafacil.com/review

Send us a photo or screenshot of your review to contacto@andafacil.com or WhatsApp +526622389950 and we will send you your gift.