ESPAÑOL (AMERICA LATINA)

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ENGLISH (NORTH AMERICA)

ANDAFACIL UPRIGHT WALKER USER
MANUAL

ANDAFACIL UPRIGHT WALKER USER MANUAL

WARNINGS

Before making any use of the walker, make sure that the walker is in good condition. In case the product is damaged, do not use it. Users are responsible for the use of the upright walker.

DESCRIPTION OF THE ANDAFACIL WALKER

The Andafacil walker has a compact and portable design made of resistant aluminum. The walker has adjustable arms and is designed to support people with limited mobility to walk upright.

COMPONENT MATERIAL

Frame: Aluminum
Tires: Rubber
Handles: Polyurethane
Seat: Nylon and Polyester

PREPARATION FOR USE

Make sure to assemble the walker correctly and, if necessary, ask for the support of a doctor or nurse to adjust the appropriate height for the user.

SECURITY INSTRUCTIONS

1. To ensure user safety, it is necessary to follow the instructions below.
2. Make sure all 4 tires are well adjusted and have air.
3. Make sure the height adjustment knobs are secure.
4. Make sure the walker is fully extended.
5. The backrest of the walker is designed to provide support when sitting in the walker. It is not made to support 100% of the body weight.
6. The brakes must be on before using the seat.
7. The brakes are designed to control walking speed but are not designed to stop the walker 100%.
8. Do not use the walker as a wheelchair.
9. Do not use the walker to walk backwards or on stairs.
10. Do not use the walker seat to transport heavy objects.
11. Do not use the walker seat on slopes.
12. Do not push the walker while the brakes are on.
13. Do not put your hands in the area where the walker bends to avoid injury.
14. Do not slump down on the walker seat. Use support arms to slide.

COMPONENTES

1. Handle
2. Back
3. brake cable
4. Shopping bag
5. 30CM front tires
6. quick release screw
7. 25CM rear tires
8. stick bra
9. Support pedal against obstades
10. Height adjustment button
11. Support handles for standing/sitting
12. handle adjuster
13. arm support

TECHNICAL SPECIFICATIONS

Distance between handles	50CM / 19.7 in
Total width	62CM / 24.4 in
overall depth	80CM / 31.2 in
seat area	25CM*46CM / 9.8*18 in
seat height	59CM / 23.2 in
Adjustable height of handles	133CM*146CM 52.3-57.4 in
maximum user weight	140KG / 300 lb
Weight	13KG / 28.5 lb
front tires	30.5CM / 12 in
rear tires	25.4CM / 10 in

ANDAFACIL UPRIGHT WALKER USER MANUAL

The Andafacil walker needs to be assembled before use. Please follow the instructions

1. Take out all the parts from the box

2. Remove the quick release bolt and install the front wheels

3. Remove the quick release bolts and install the rear wheels.

3. Remove quick-release screw and install rear wheel then tighten screw.

4. Unfold the walker and make sure the wheels are in the correct position.

5. Locate the 2 handles and insert them into the front of the walker tubes. Use the buttons to adjust the height.

6. Insert the arm supports in the indicated position.

7. Locate the support handles and insert them into the front of the walker.

8. Hang the bag on the tubes in front of the seat made to hold the bag.

ENGLISH (NORTH AMERICA)

HOW TO ADJUST THE ARM SUPPORTS

Use the buttons along the arms of the walker to adjust the height. Press the button to slide the arm supports up or down.

HOW TO FOLD THE WALKER

To fold the walker, you must pull the strap that is on top of the seat. Remove the quick release bolts and fold the blades.

HOW TO USE THE BRAKES

Squeeze the brake to slow or stop the walker. Push the brake Forward until you hear a click to put the walker in the stationary brake.

Attention: the brakes must be in stationary mode to sit on the seat.

MAINTENANCE

Over time the brakes may need adjustment. If the cables feel loose, adjust them the same way you adjust the brakes on a Bicycle. Make sure brakes and handgrips are properly always adjusted.

Maintenance must be carried out as follows:

WEEKLY

- Check the operation of the brakes
- Check that the tires are tight

PERIODICALLY

- Check the handles
- Make sure the screws are tight
- Make sure the tires are tight

ANDAFACIL UPRIGHT WALKER USER MANUAL

CLEANING

The Andafacil walker should be cleaned with a damp cloth. Use a disinfectant cleaner if required. The walker must be completely dried before using it again to avoid operating risks.

WARNINGS

1. Maximum user weight: 140KG
2. Do not stand on the walker
3. Make sure the handles are tight
4. Seek support from a medical professional if assistance is required to adjust to the proper height
5. Always keep the distance between the user of the walker and other people.
6. Do not attempt to replace the wheels or brakes on the walker. If replacement is required, contact Andafacil.
7. Make sure the walker is fully extended before using it.
8. Avoid putting your hands in when folding the walker.
9. Make sure the tires have enough air.

Thanks for your purchase. We hope you enjoy your Andafacil walker.

Support us with a rating on the platforms and networks and you will receive a special gift from us

Visit the page or scan the QR code to get started

www.andafacil.com/review

Send us a photo or screenshot of your review to contacto@Andafacil.com or WhatsApp +526622389950 and we will send you your gift.