

Multi-function nursing bed instruction manual

one. Product name: multi-function nursing bed

two. Specifications: CBC-02

three. product performance, structure and composition:

multi-function nursing bed is mainly composed of foot-frame movable bed, foot-frame guardrail, foot-frame lifting bracket, foot-frame control handle, foot-frame lifting rocker handle, leg-frame, breech-position bed, back-frame, back-position strut, defecation hole, bedpan carrier closing screw drive mechanism, defecation rocker handle, caster, brake gate, living table plate, bed frame, backframe screw, infusion hanger, side guardrail, side-turning adjustable bed plate, side-turning handle, side-turning driving mechanism, widening adjustable bed surface, widening adjustable positioning pin, aluminum bed frame, aluminum bed frame, wooden bed frame, wooden bed frame, washbasin, bedpan, drainage thread pipe.

four. Scope of application: it is applicable to the rehabilitation treatment and nursing of patients who have lost the ability of self-care in medical institutions and families.

five. Main technical data: (in MM)

Basic size (length x width x bed height)v	Size of opening of toilet hole	The location of the toilet hole
TH01A-1 2000X 960X 500	270X200	The center of the toilet hole is 950mm away from the head of the bed
TH02A-1 2000X 960X 500	270 X200	The center of the toilet hole is 950mm away from the head of the bed
TH01B-1 2000X 960X 500	270 X200	The center of the toilet hole is 950mm away from the head of the bed
KL-DHC2 2000X 960X500	270X 200	The center of the toilet hole is 950mm away from the head of the bed

six. Adjustment range of bed frames:

Name	The back frame is angled	Thigh position bed surface	The foot frame is angled	Roll over and angle up
Technical parameters	0-85°	0-30°	——	——

seven. The composition of the structure is as follows:



eight. Installation:

1. Installation of Casters: four casters have two with the brake brake, the brake with the two casters placed diagonally in the back frame-side of the bed legs of the screw hole, then install the remaining two casters in the other two bed leg angle screw hole, tighten can be.
2. The installation of the back frame: the back frame and the bed frame can be tight with the back frame pin.
3. Foot frame installation: the foot frame with the foot frame pin pin can be tight. The head of the bed is double-lock plug-in type. With anti-tripping film, the big one is the head of the bed, the small one is the end of the bed, please pay attention to distinguish.

nine. use method:

- 1, Posture Adjustment: grip the back frame control handle shake hard, drive the back frame slowly rise, rise to the required angle when the release of the back frame control handle, the same grip back frame handle in the opposite direction to shake down, the leg frame by the leg frame handle control, foot frame by the foot frame control handle control. Can make the patient obtain from the supine to the semi-supine, bent leg, sitting and other postures.
2. The use of defecation device: turn the defecation handle to open the defecation hole cover automatically. At the same time, the carrier of the urinal is automatically sent to the patient's buttocks, the patient can urinate or defecate or wash the lower part, turn the handle of the urinal counterclockwise to close the urinal hole and keep it level with the bed surface, and the urinal is automatically sent out, and the nursing staff takes it to wash, after cleaning, put the bedpan back into the carrier for next use.
- 3, side guard rail installation: side guard rail and bed frame on the guard rail hole alignment, can be fastened with bolts.
4. Installation and use of the living table board: The Living Table Board is provided with 2 concave positioning plastic parts on the back of the living table board, which can be buckled into the top of the side guardrail.
5. Installation and use of the infusion hanger: when using the infusion hanger, only need to insert the positioning pin on the infusion hanger into the round hole above the side guard rail, reverse operation can be removed.
6. brake installation use: with the foot or hand under the pressure brake brake brake for the brake, lift up for release.

ten. Note:

1. When choosing to change function or wheelchair function or let the patient sitting defecate, you need to raise the back frame, pay attention to before it is necessary to raise the leg frame to the right angle, this prevents the patient from sliding down the leg frame as his head rises.
2. Don't Park on a ramp.
3. often check the activities of the pin shaft, screws, rails on the wire to prevent loosening, falling off.
4. Add a little oil to the screw nut and hinge shaft every year.
5. lead screw and other parts, please do not force the operation, failure should ask professional maintenance.
6. When you wash your hair, water temperature please do not exceed 40° c.
7. foot frame up and down, please first light foot frame, and then lift the control handle, to prevent broken handle.
8. keep clean appearance, spray surface do not scratch or hit."

eleven. Transportation:

packaged products may be transported by general means of transportation. In the course of transport should pay attention to anti-slip, rain and snow, not with toxic, harmful or corrosive substances mixed transport.

twelve. Storage:

B packaged products should be stored in the relative humidity not exceeding 80% , non-corrosive gases and well-ventilated indoor.

Thirteen. This product warranty-year.

Fourteen. Packing and packing list:

Carton Packing, packing of product parts and packing list.

Installation diagram of multifunctional nursing bed

Nursing bed

Instruction manual

Please take good care of this product instruction manual so that you can read it at any time and read it carefully before using the product



Lie flat function



Infusion function



Lengthen the guardrail



Help move the mute wheel

