ANDAFACIL

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Andafacil® Andadera Bariátrica Extra Resistente

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SKUs: AF4R25-ER-R · AF4R25-ER-B

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ENGLISH (NORTH AMERICA)

Andafacil® Extra-Sturdy Walker with Seat
AF4R25-ER-R & AF4R25-ER-B

WARNINGS

Before carrying out any operation, visually check the integrity of the device. Where the product presents anomalies, immediately contact the dealer from whom it was purchased. We assumes no responsibility for the misuse of the device.

DESCRIPTION WALKER

The Walker features a compact folding design, aluminum stable frame and fully adjustable handrails designed to help users have more outdoor movement for exercise and leisure purposes.

MATERIAL COMPONENTS

Frame: Aluminum
Wheels: Solid tyre
Handrail: Non-slip textured handrails
Seat: Black leather seat cushion

PREPARING FOR USE

The choice of the type of aid and the necessary settings for use must be carried out by medical and technical staff qualified, depending on the pathology and user terms and conditioning.

SAFETY INSTRUCTIONS

1. In order to ensure your safety when using the walker, you must comply with the following safety information and all instructions.
2. Check that all wheels are tight before use.
3. Check whether the height adjustment button is secure.
4. Make sure your walker is in place. Be aware of your surroundings when using a walker.
5. The purpose of the backrest is to provide back support when sitting down. The backrest is not designed to support the whole body weight. Avoid leaning while sitting, as this may cause you to fall.
6. The brakes must be in a locked position before sitting in the seat.
7. Brakes are used to prevent wheels from rolling. However, if someone pushes it, it may not prevent the walker from moving.
8. DO NOT push your walker while sitting. Walker did it on purpose.
9. DO NOT use a walker to go backwards, down slopes, climb stairs, curb, or cross large obstacles. There may be a serious risk of falling or injury.
10. DO NOT use seats to transport people or large objects.
11. DO NOT sit in your seat when the walker is on an incline.
12. DO NOT push the walker when the brake is on.
13. DO NOT put your hands on the folding of the walker to avoid getting caught.
14. DO NOT allow yourself to fall on the seat of the walker, but always sit slowly in the seat in a controlled manner.

COMPONENT

1. Hand grips
2. Hand brake lock
3. Screw
4. Seat
5. Rear wheel
6. Padded backrest
7. Brake cable
8. Storage bag
9. Universal wheel

TECHNICAL SPECIFICATIONS

Walker width	21.06in
Walker length	30.9in
Seat height	19.68in
Seat width	10.24in
Seat length	19.48in
Adjusted max height	34.65in
Adjusted min height	30.71in
Max. User's weight	300lbs
Weight	18lbs
Front wheel	6in
Rear-wheel	6in

The walker requires assembly before use. Please follow the instructions below:


Unfold the main frame

.Insert the rear wheels into the main frame(noted that the wheels mustbe facing outward)


Insert scew to the hole and install the gasket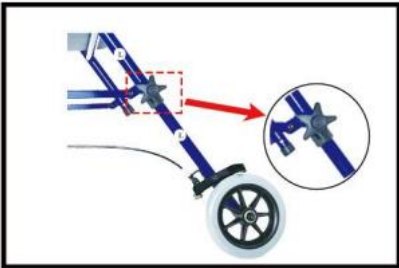

Use rotating nut to screw it tight

This part shall be facing outward,insert the front wheels to main frame

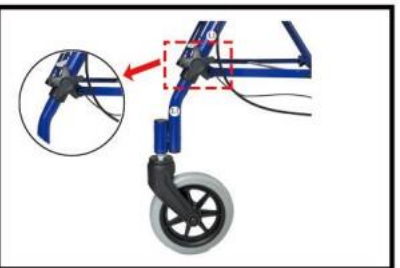.Screw it tight

.Insert the adjustable handle into main frame (note that the brake lines correspond tothe left and right)

.Screw it tight

Using the cable clip to fix the cable properly

Place the back into the main frame


Press button and insert into the frame

.Completion

HOW TO ADJUST THE HEIGHT?

Turn the screw and take it out, adjust it to the right height, and lock the hole position.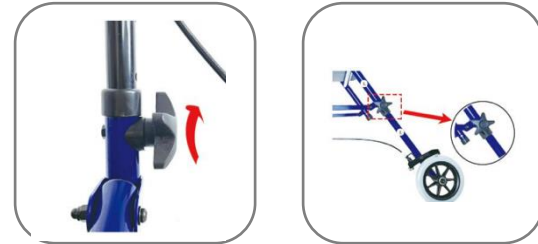

FOLDING THE WALKER

To fold a walker, simply lift the crossbars on either side


1. Pull it up from the middle, then you can fold it

OPERATING THE HAND BRAKES

Hand brake: Hold the handbrake to slow or stop the Walker.

Fixed brakes: Pull the brakes outward until they "click" to lock the walker in position to lock the rear wheel.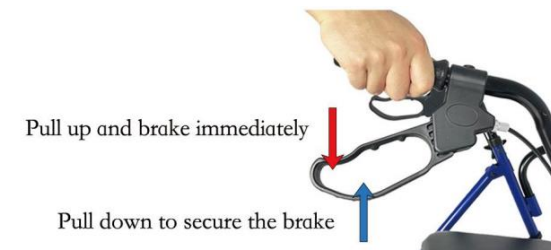

MAINTENANCE

Over a period of time the brake cables may stretch, and if no further adjustments can be made the cables should be replaced. Seek the help of your dealer when replacing brake cables.

The castors should be checked that they are secure each time before using the walker. Also, check the brakes operate correctly and the handles and hand knobs are secure. Should any part of the walker appear loose or insecure discontinue use until it is inspected by an authorized dealer.

The following shows the maintenance schedule:

WEEKLY

- Check brake for malfunction
- Check all wheels and castors for loose bolts and nuts and knobs

PERIODICALLY

- Check handgrips for wear
- Check frame bolts are secure
- Check front and rear axle bolts are secure
- Check tyres for wear or cracks

YEARLY

- Once a year take your walker to an authorised repairer for a full check, clean, service and lubrication of all moving parts

CLEANING

Your walker can be cleaned with a damp cloth or sponge and warm soapy water. Domestic anti-bacterial surface cleaners may be used if required. Care should be taken not to leave any cleaning residues on the handgrips or seat which may leave them slippery when used. Do not use scouring pads or scrapers as this may damage surface coatings.

WARNINGS!

1. Maximum working limit 300 LBS.
2. Don't stand on the product
3. Make sure both arms are adjusted to the same height.
4. Please consult a trained healthcare professional or an approved freelance healthcare distributor for height adjustment of this product.
5. Keep a safe distance between pedestrians and other pedestrians or objects.
6. Some parts may cause injury or damage when they come into contact with other objects.
7. Do not attempt to change the wheels or casters of your bike. If they are faulty or need to be replaced, contact an authorized repairman.
8. Make sure the walker is fully open (the folding bars on both sides are straight) before using it.
9. When folding, keep your fingers away from the folding bar.
10. Make sure the brakes are working properly before use

Thanks for your purchase. We hope you enjoy your Andafacil Walker.

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